



No mesh. No grafts. Fascial closure in 7 days*.

Case photo courtesy of Dr. Fernando Ferreira, Hospital Pedro Hispano, Matosinhos Portugal. (Perforated Silicone Sheet shown in situ)

ABRA® Abdominal Wall Closure

- > Restores domain
- > Primarily closes fascia and skin
- > Reduces OR procedures by 50%*
- > Saves \$12,000 to \$47,000/patient*
- > **92% primary closure rate***



ABRA® Abdominal Wall Closure
Because nothing beats a primary closure.

www.justcloseit.com

ABRA® systems:

- > **Abdominal Wall Closure**
- > Surgical Skin Closure
- > Adhesive Skin Closure

CANICA® 1-800-705-8312
1-613-256-0350
www.canica.com

*Early Primary Closure of Open Abdominal Wounds Using the Abdominal Reapproximation Anchor (ABRA®) System.
Cinelli SM, et al. UMC Southern Nevada, Las Vegas, NV. Poster: Southwestern Surgical Congress, March 2009

ABRA® Abdominal Wall Closure A Dynamic Wound Closure System

What it does:

ABRA® Abdominal Wall Closure enables primary closure of both fascia and skin.

Indications:

ABRA® Abdominal Wall Closure is indicated for retracted, full-thickness midline abdominal closure after laparotomy for Abdominal Compartment Syndrome (ACS), abdominal hernia, mesh removal, AAA, or abdominal trauma, and for retention of abdominal wall closure.

Why it is used:


Benefits:

- Achieves a low-tension primary closure
- Maintains domain, or quickly restores lost domain
- Eliminates hernia and need for mesh and skin graft
- Preserves fascial margins
- Restores normal physiology

Features:


- Used with Negative Pressure Wound Therapy (NPWT)
- Reduces OR procedures by 50%
- Reduces length of stay
- Allows bedside dressing changes
- MRI compatible

Key System Components:

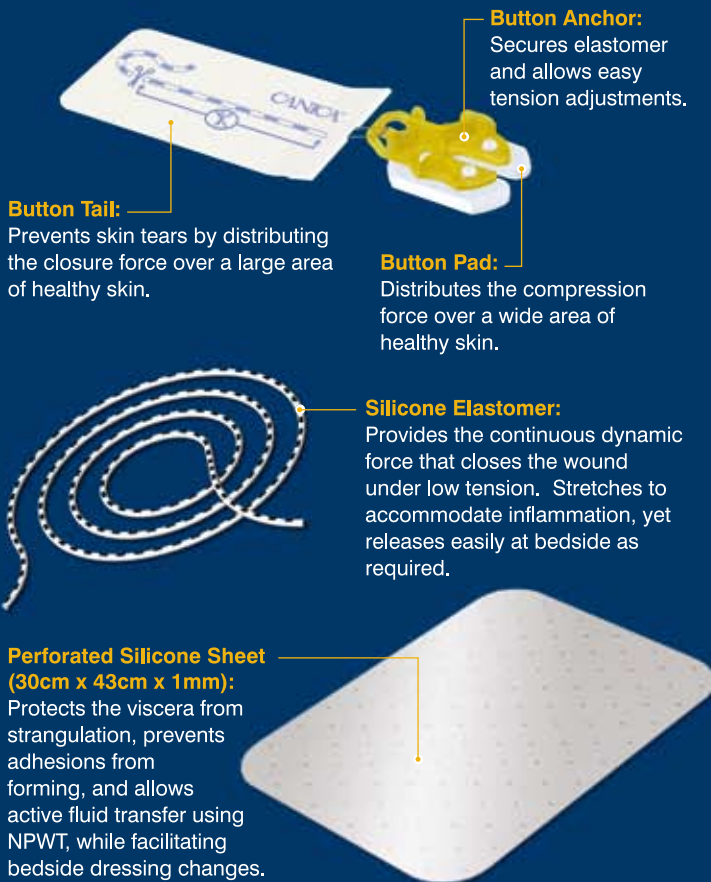

How it works:

The ABRA® Abdominal Wall Closure System restores the primary closure option for full-thickness, retracted midline abdominal defects. This Dynamic Wound Closure System pulls the muscle planes together under low tension while leaving fascial margins intact and ready to suture, for a sound primary closure. This ABRA system eliminates the hernia without the need for mesh, and re-approximates the skin margins, eliminating the need to graft.

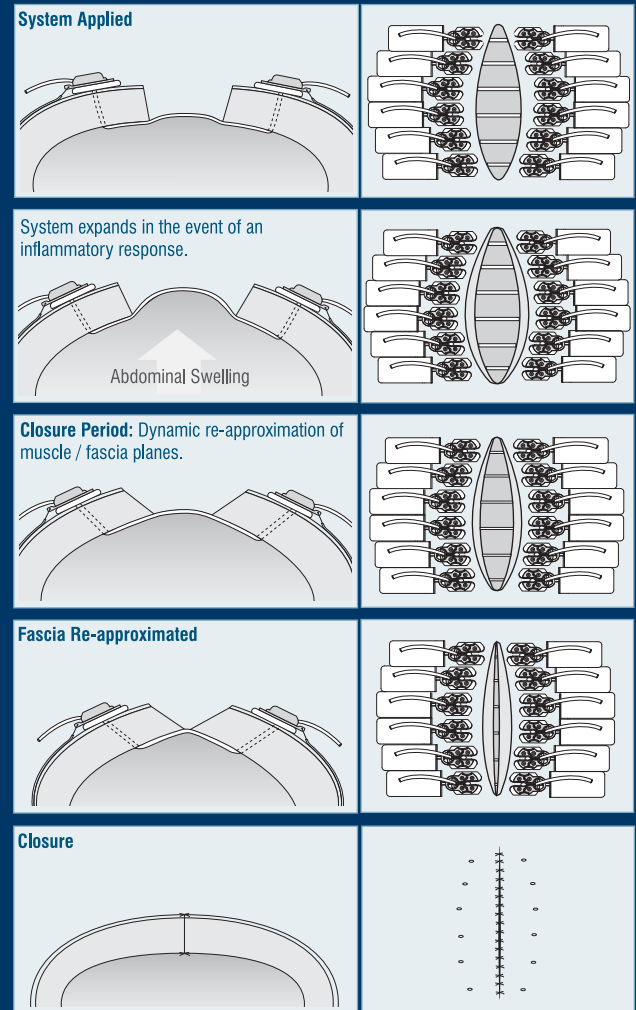

Ordering Information

Description	Part Number
ABRA® Abdominal Wall Closure Set	CWK08
ABRA® Extension Kit (for wounds wider or longer than 30 cm)	CWK08-EX

US Patent Nos. 7,361,185 & 7,429,265. Other International Patents.

ABRA® Abdominal Wall Closure

Because nothing beats a primary closure.



For your local distributor and product ordering information, please contact Customer Service at +1.613.256.0350, toll-free at 1.800.705.8312 (USA and Canada) or online at www.justcloseit.com.