## Prepare la piel que rodea la herida.

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To learn more about SutureSafe, visit:

**www.suturesafe.com**

To re-order SutureSafe call **Lee Valley Tools:**

**Canada: 1-800-267-8767**

**USA: 1-800-871-8158**

For product enquiries call **Canica Design, Inc.:**

**1-800-705-8312**

Do not reuse.

Product and packaging are 100% latex free.

**STERILE R**

Available sizes	Part Number
<b>SutureSafe</b> , 1/2 in x 4 in (13 mm x 102 mm), 12 /box*	<b>DWS1212</b>
<b>SutureSafe</b> , 5/8 in x 8 1/2 in (16 mm x 216 mm), 8 /box*	<b>DWS1408</b>

\*Either product will cover 2 applications on a 4" - 4 1/2" (10-11 cm) sutured wound.

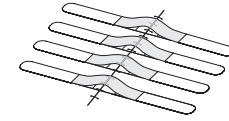
**Canica Design Inc.**  
**Head Office**  
 36 Mill Street, Almonte, ON Canada K0A 1A0  
**U.S. Distribution Center**  
 814 Proctor Ave, Ogdensburg, N.Y. USA 13669-2205

**Phone:** 1-613-256-0350  
**Fax:** 1-613-256-0360  
**Email:** sales@canica.com  
**Web:** www.canica.com  
**Toll Free:** 1-800-705-8312

**CANICA®**

Questions?  
**1-800-705-8312**

or call 1-613-256-0350 TRI-1D1456RA

## Instructions for Use Directives d'utilisation Instrucciones de uso

### How does SutureSafe work ?

English

SutureSafe is designed to reduce tension and pain, prevent re-opening of sutured wounds, and to close small cuts. It has a special elastic center that gently pulls wound edges together to support the wound. The transparent RipStop™ prevents the wound from pulling apart (especially during sudden movements such as coughing and sneezing). At all other times, the elastic center reduces tension on the wound by creating an area of relaxed skin adjacent to the incision line, reducing incisional pain and scarring, and encouraging mobility.

#### Warnings

If an allergic reaction occurs, discontinue use. SutureSafe is for use on healthy, sutured wounds of any length, or small cuts. If the wound/cut is infected, see a doctor before use.

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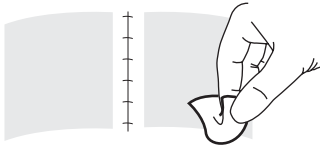
1


**Prepare skin around the wound.**

**SutureSafe sticks best to clean, hairless skin.** Wash skin with soap and water, and dry. Wipe skin with rubbing alcohol to improve adhesion. If necessary, remove body hair by shaving.

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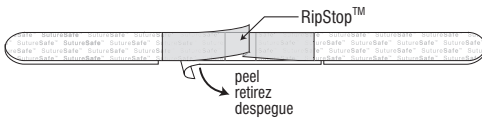2


**Peel off bottom center liner.**

**Note:** Avoid touching the exposed surface.

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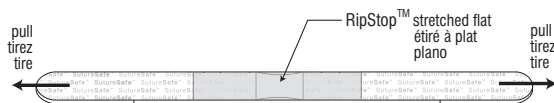
3

**Gently stretch until RipStop™ becomes just flat.**

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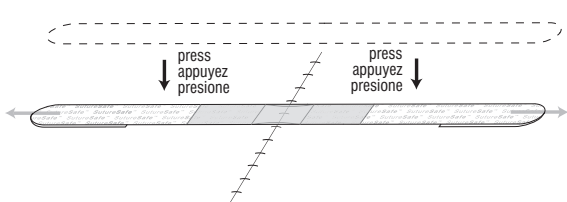4

**Center over wound, then press on skin.**

Center SutureSafe as shown and press to skin on both sides at the same time.

**Note:** For difficult to reach areas, ask for help in applying.

**Warning:** Do not pull the skin as this can cause skin damage or failure of SutureSafe to stick to skin.

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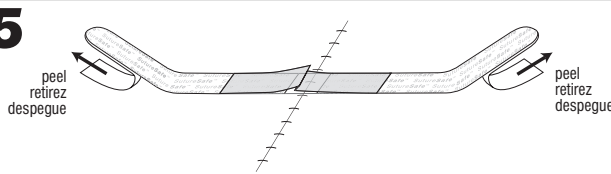
5

**Peel off outer liners and apply.**

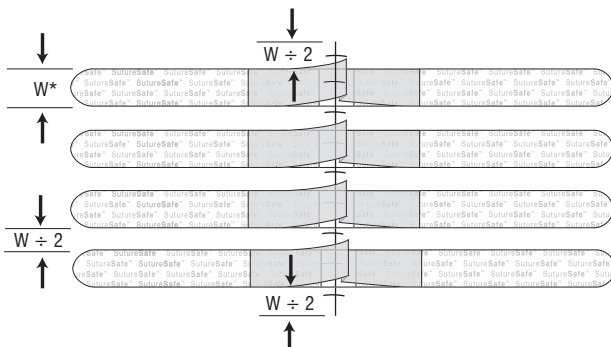
Rub fabric down gently to ensure SutureSafe sticks well to skin.

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\*W = width / largeur / anchura

**Repeat steps 2 - 5 to cover wound or cut.**

Place SutureSafe along entire length of the wound or cut, as shown. Maximum spacing between SutureSafes should be half the width of the product, as shown.

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## Using SutureSafe

If necessary, dressings can be applied on top of SutureSafe. SutureSafe is shower friendly. Let SutureSafe air dry prior to covering with clothing or dressings.

### When to replace

Apply fresh SutureSafe as needed. Each SutureSafe can last up to seven days.

### How to remove

Peel ends back toward the wound for painless removal.

**Note:** A sticky residue may be left on skin if SutureSafe is removed wet. To remove residue from skin, let dry, then rub off.

### When to Stop

SutureSafe is recommended for use over sutures/staples/glue for a minimum of two weeks. If you are using SutureSafe to close a small cut, apply until cut heals (usually about 10 days). Longer use (up to 29 days) can further reduce scarring.

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